

Instructions for use SYNMAN Compact Torso Model

SYNBONE®
better education outcome

Product overview / SYNMAN Compact Torso Model and Accessories



SYNMAN Compact case with attachable wheels for easy transportation



SYNMAN Compact Torso Model for surgery trainings with simulated organs



SYNMAN Compact parts available individually or in sets for 8, 16 or 20 participants

SYNMAN Compact Torso Model consists of (1) piece each of below components:

1. Cricoid
2. Skin Cricoid
3. Pad Chest
4. Skin Chest
5. Pad Rib right
6. Pad Rib left
7. Skin Rib
8. Food coloring red
9. Food coloring light red
10. Food coloring yellow

Attach the Pad Chest



Open the SYNMAN Compact Torso Model by pressing the center button of the hinge clasps



Replacement parts:
PR0624.03 Pad Chest
PR0624.04S Skin Chest



The white plate can be opened by removing the black screw



Place the Pad Chest into the opening and re-cover with the white plate. Tighten the black screw



Once the Pad Chest is securely in place, the outer skin can be reapplied

Insert the cricoid



Replacement parts:
PR0624.01 Cricoid
PR0624.02S Skin Cricoid



Remove the Skin Cricoid



Place the Cricoid into the hole



The Skin Cricoid can be reapplied

Instructions for use SYNMAN Compact Torso Model

SYNBONE®
better education outcome


Insert the lungs



Replacement parts:
PR0624.12L Lung left
PR0624.12R Lung right



Clean the lung surfaces with a small amount of acetone and sprinkle with talcum powder

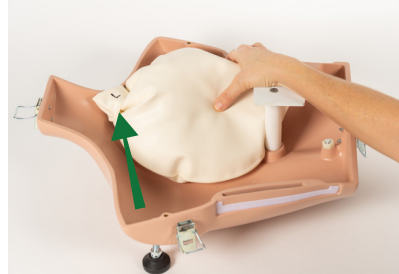

Insert the left lung and tuck it securely in place with a smooth outer surface




Insert the right lung and tuck it securely in place with a smooth outer surface




The top of the Torso Model can be placed on the lower portion and the side hinge clasps can be closed

Insert the Pads Ribs

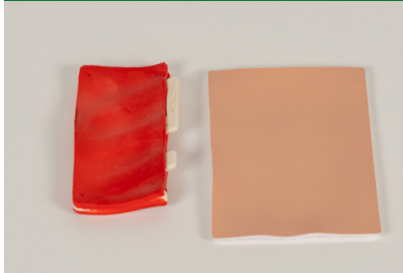


Replacement parts:
PR0624.05 Pad rib right (.06 Pad rib left)
PR0624.07S Skin rib

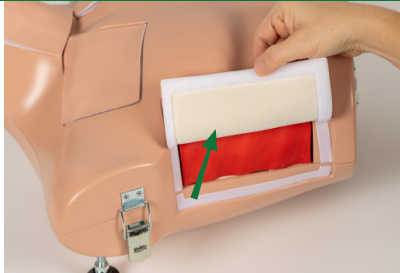

Remove the outer skin on both sides of the Torso



Insert the Pad rib right and Pad rib left on both sides (the Pads are marked right and left)

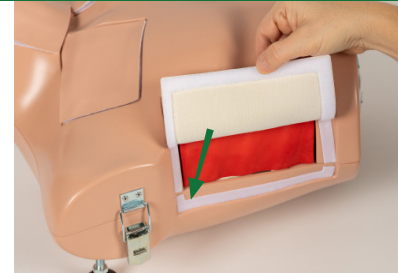

Reapply the outer skin



The SYNMAN Compact Torso Model is now ready for use

Instructions for use

SYNMAN Compact Torso Model



Intended use

The **SYNMAN Compact Torso Model** is specially designed for teaching three principle Advanced Trauma Life Support surgical procedures. The SYNMAN Compact Torso Models were designed in cooperation with ATLS Switzerland and are officially acknowledged by ACS/ATLS. For many years, the manikins are being used in acute trauma life support courses throughout the world. The **SYNMAN Compact Torso Model** is not intended to be used in the operation room.

SYNBONE® guarantees best results with the **SYNMAN Compact Torso Model** only by using the **SYNBONE® Accessories for SYNMAN Compact**.

Cleaning and storage

Clean the **SYNMAN Compact Torso Model** with a damp cloth after each use. Do not use any acidic cleaning products.

To guarantee a long life of the **SYNMAN Compact Torso Model** we recommend a storage in the **PR1221.90 SYNMAN Compact Travel Case**.

Accessories

PR1221.90	SYNMAN Compact Travel Case	PR0624.01	SYNMAN Cricoid
PR1221.51	SYNMAN Compact Part Set 8	PR0624.02S	SYNMAN Skin Cricoid
PR1221.50	SYNMAN Compact Part Set 16	PR0624.03	SYNMAN Pad Chest
PR1221.52	SYNMAN Compact Part Set 20	PR0624.04S	SYNMAN Skin Chest
		PR0624.05	SYNMAN Pad Rib right
		PR0624.06	SYNMAN Pad Rib left
		PR0624.07S	SYNMAN Skin Rib
		PR0624.12L	SYNMAN Lung left
		PR0624.12R	SYNMAN Lung right
		PR1236	SYNMAN Talcum Powder

Contact us

SYNBONE AG
Tardisstrasse 199
7205 Zizers · Switzerland
Phone +41 81 300 02 80
sales@synbone.com · www.synbone.com

SYNBONE SDN BHD
No. 40 - 41, Jalan i-Park 1/3
81000 Bandar Indahpura Kulai, Johor · Malaysia
Phone +60 766 082 20
sales@synbone.com · www.synbone.com

