

# Instructions for use RAY Knee Holder for Distal Femoral Osteotomy Training

**SYNBONE®**  
better education outcome

## Optional set up to RAY Knee Holder System for Distal Femoral Osteotomy (DFO) Training



The RAY Knee Holder System can optionally also be used for DFO training



0027.35 RAY Knee Holder adapter set for DFO training



LD2172 Femur malformation notch with mounted adapter set 0027.35



Remove the standard notch holders from the brackets

## Adjustment



Insert the first DFO adapter into the hole of the femur bone model



Insert the second DFO adapter into the notch of the femur bone model



Insert the DFO adapter to the bracket of the guiding rail and tighten the locking lever with slight pressure




Insert the DFO adapter to the bracket of the supporting column and tighten the locking lever with slight pressure




The RAY Knee Holder can now be used for DFO training

## Distal Femoral Osteotomy (DFO) Training

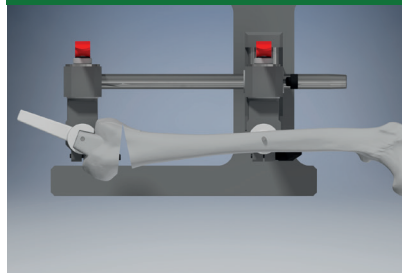

Visualisation of closed wedge technique training with RAY Knee Holder System

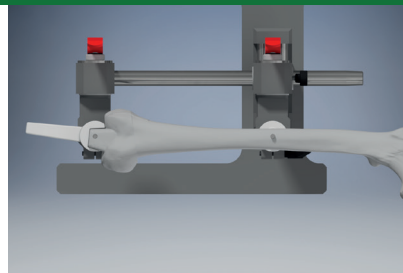

Visualisation of closed wedge technique training with RAY Knee Holder System

## Contact us

### SYNBONE AG

Tardisstrasse 199  
7205 Zizers · Switzerland  
Phone +41 81 300 02 80  
sales@synbone.com  
www.synbone.com

### SYNBONE SDN BHD

No. 40 - 41, Jalan i-Park 1/3  
81000 Bandar Indahpura Kulai, Johor · Malaysia  
Phone +60 766 082 20  
sales@synbone.com  
www.synbone.com