



Instructions for use

Bone Clamp With Ball Head


Product function / Overview

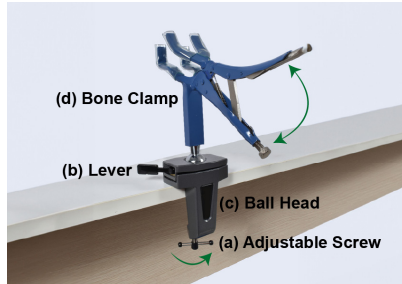

Place the ball head bone clamp on the table, tighten the screw firmly. Use the lever to lock/unlock the ball head.

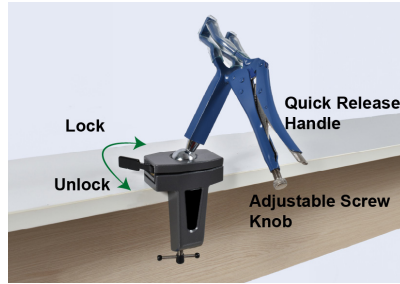

Use the lever to lock/unlock the ball head.

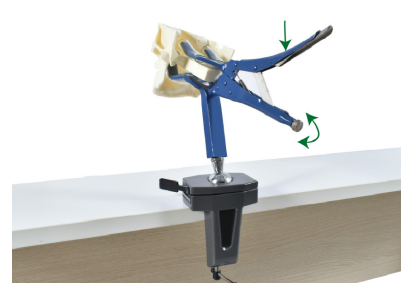

Ensure the clamp is open. Rotate the screw knob counterclockwise. Place the bone, then close the handle.



Rotate the screw knob clockwise till both jaws of the bone clamp are touching the bone.




Open the clamp, rotate knob clockwise to increase tension, secure the bone. If loose, adjust knob for right tension.

Good to know : Setting mobility of the ball head



Ensure the lever is in the "unlock" position.

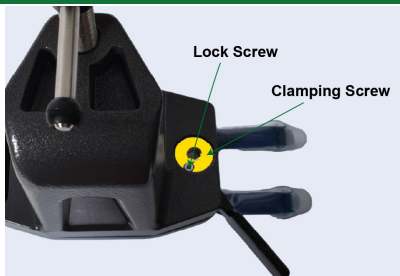

Lock screw (Small); clamping screw (Big)



Unlock the ball by using lever. Remove the lock screw.



Turn the clamping screw to next hole in the corresponding direction.



Afterwards tighten the lock screw in one of the 9 intended holes

Cleaning and storage

Clean the **Bone Clamp with Ball Head** with a damp cloth after each use. No lubrication is necessary.

Accessories

1. Bone Clamp
2. Ball Head
3. Allen Key, size 5.0mm

Contact us

SYNBONE AG
Tardisstrasse 199
7205 Zizers · Switzerland
Phone +41 81 300 02 80
sales@synbone.com · www.synbone.com

SYNBONE SDN BHD
No. 40 - 41, Jalan i-Park 1/3
81000 Bandar Indahpura Kulai, Johor
Malaysia
Phone +607 660 8220
sales@synbone.com · www.synbone.com