



Instructions for use ROB Knee Holder Rail System

Product Overview

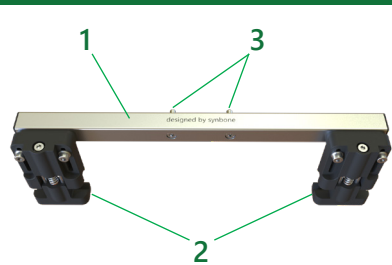


Rail Adapter components

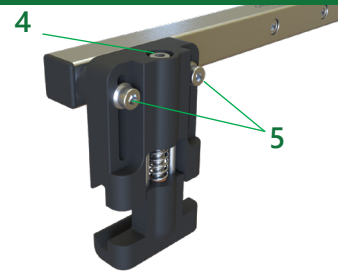


Rail Clamp components

1. Connector bar
2. Rail Clamp
3. Attachment screws
4. Clamping screw
5. Height adjustment screws


Assembling and Rail Attachment (The following is for use with left-sided knee models)

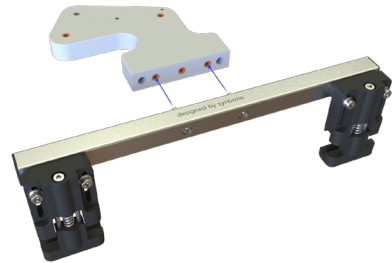


Attach the Rail Adapter to the Base Plate...

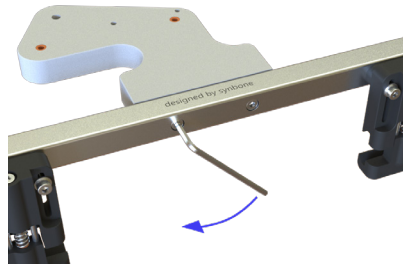


... and tighten the attachment screws (3) with the Allen Key

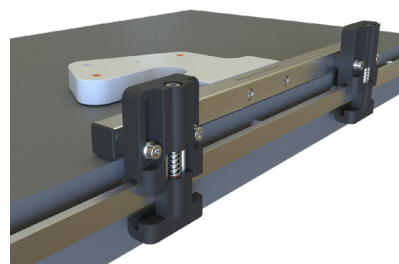

Make sure that the height adjustment screws (5) are loose and attach both Rail Clamps (2) to the rail

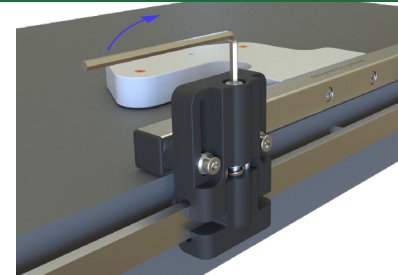


Tighten the clamping screw (4) until the connection to the rail is stable. Repeat the step for the second clamp

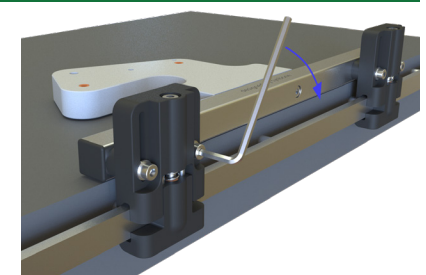

Confirm that the Base Plate is lying on the table and tighten the height adjustment screws (5) on both clamps

Assembling

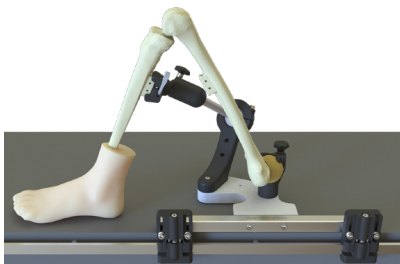

Continue with procedures described in the ROB Knee Holder System instructions



To use the ROB Knee Holder System with right-sided knee models the Base Plate has to be flipped by 180°